

USER MANUAL SUPPLEMENT




This is an updated user instruction for guaranteeing adequate battery charge for patient exposures.

When changing the battery always use a fully charged battery, fresh from the charger.

If you are unsure regarding the state of charge for the battery, follow the procedure below.

Checking for Adequate Battery Charge

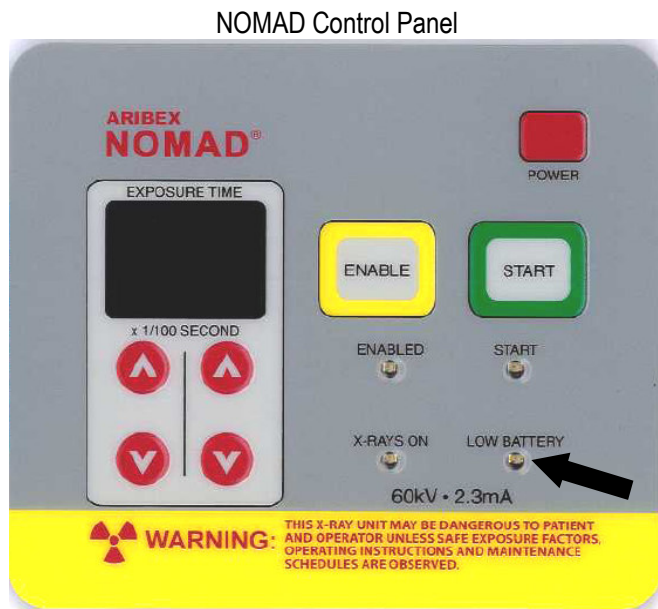

❶ After inserting a battery, turn on the power. The numeric (LED) display should illuminate and you should hear a single, long audible \ll signal (if not, the battery is fully depleted).

❷ Check for adequate battery charge by making a non-patient test exposure in a manner that will not result in bystander or operator exposure.

❸ If the battery is too low, the yellow **LOW BATTERY** indicator \odot at the lower right of the control panel will flash (accompanied by an intermittent audible \ll signal). The battery **MUST** be recharged before use.

The **absence** of the **LOW BATTERY** indicator alarm (on the lower, right-hand corner of the NOMAD Control Pane) is indication of adequate operating charge to take an X-ray. If this alarm illuminates, change to a charged battery before attempting additional exposures.